



Topi-Click® Dispensing Instructions


Read all information and instructions carefully before dispensing from the Topi-Click™.

1. Pop off the cover by placing thumb in groove on the front where it says REFILL and press upward on the cover's thumb tab.
2. Turn the base slowly as shown below in Fig. 2 to dispense the specified _____ # of clicks per topical provider's instructions. You may refer to the bottom for reminder instructions.
3. Apply by rubbing applicator pad to area of skin as directed by topical provider.
4. Replace cover firmly and tightly.
5. When the bottom line (see Fig. 5 below) reaches the engraved line above the word "Refill," it may be time to call in for any refills because there are approximately 12 clicks left.

The Refill Indicator: There are approximately (depending on viscosities) the following numbers of clicks left when the very bottom of the topical elevator reaches the line engravings on the front; Refer to Fig. 5 below. Look at the top front for these engravings:

_____ = 12 clicks left (first line from the top)
Refill

_____ = 32 clicks left (second line from the top)

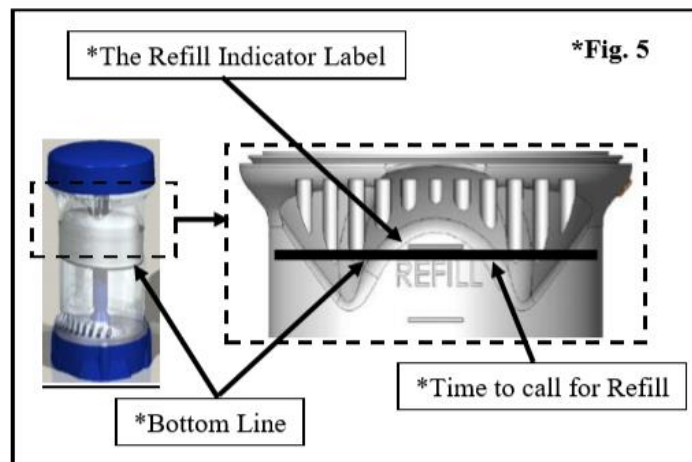

The Topi-Click® is a one-time use container and not refillable.

Downloadable at www.Topi-Click.com

Downloadable at www.Topi-Click.com

The Topi-Click is a one-time use container and is not refillable.

The Topi-Click® product may be improved over time based on customer feedback on usage. Custom Rx TDA, LLC and/or the patent holders are not liable for any damages arising from the products dispensed in the Topi-Click® or its improper use.